

New FamoScope Endoscope cleaning Units




Transport Trolleys



Height adjustable worktables



Height adjustable, fully automatic
precleaning station with
ultrasonic cleaner and spraying
tank with automatic tray lift

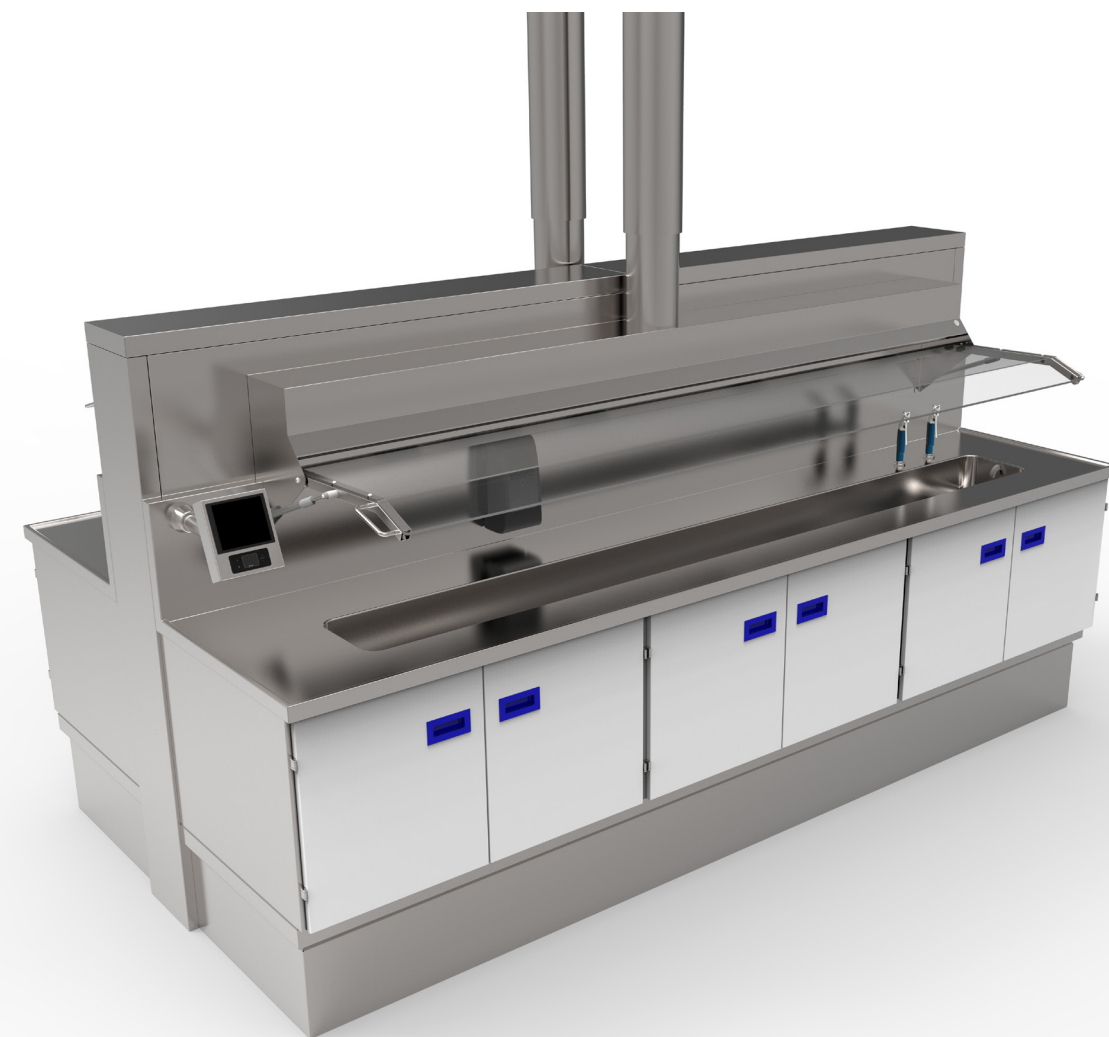

FamoScope

Perfection into Every Detail

Famos bv, Noorderpoort 20,
5916 PJ Venlo, The Netherlands,
T : +31 - (0)77-3548800,
F : +31 - (0)77-3520432,
E : famos@famos.nl,

New FamoScope Endoscope cleaning Units

Impression

The Concept:

A proper cleaning of flexible endoscopes has become more important and critical.
"IF IT ISN'T CLEAN IT CAN'T BE DISINFECTED"!

The new FamoScope endoscope cleaning unit can be adjusted to your protocols and workflow.

Options include fully automatic height adjustable cleaning unit, incorporating closed cabinets, automatic filling with preset volumes along with automatic chemistry dosing and automatic draining. Touch-screen control, spray hood, air suction and air leakage tester with LED support light.

Or the basic unit with one sink, manual filling and draining, fixed height and open or closed structure.

Famos designs it.

Famos options:

Ergonomics: To create a work environment that supports the ergonomic needs of the employee, Famos created an Endoscope cleaning unit that is completely height adjustable. Other options include an open or closed plinth under the unit.

Reproducible: A reproducible process time after time was at the forefront of the Famos design team when creating the new Endoscope Cleaning Unit. Essential to this is the touch-screen, 3 levels of water can be set into the software. The chemical pump assures the correct concentrate of chemical dosage is automatically added to the water, based on the manufacturer's DFU of your chemistry.

After the endoscope is cleaned, you can select the drain function on the touch-screen control to automatically drain your sink. An automatic flush after draining makes sure sink residues are removed.

An incorporated timer assures draining the sink will automatically occur after the inputted preset working hours. The chemical pump will give an alarm if the chemical level is too low to start a new filling.

Safety: To protect the employees the Endoscope Cleaning Unit comes with an optional sprayhood cover covering the complete sink, air suction to remove aerosols and LED lights to add vision. An integrated automatic air leakage tester is also an option.

If your requested product is not in our standard assortment, feel free to contact us and together we will find a solution.

